

FIELD CONNECTION WIRES-----18 GAGE
 POINT WIRES-----22 GAGE
 STATION WIRES-----22 GAGE
 AUDIO WIRES-----22 GAGE
 SHIELD


NOTE!
 RUNS OVER 750 FT INCREASE FIELD CONNECTION
 WIRES AND POINT WIRES ONE GAGE SIZE.

IMPORTANT

TIE TOGETHER AND TAPE AUDIO SHIELDS AT
 ALL JUNCTIONS POINTS AND CONNECT ONLY
 TO SHIELD (SH) TERMINAL IN CONTROL PANEL.


A CONNECTOR IS SUPPLIED
 WITH EACH PATIENT STATION.
 WIRES NOT USED MUST BE
 ELECTRICALLY ISOLATED

IT IS RECOMMENDED THAT A FIELD CONNECTION
 CABLE RUN FROM THE CONTROL PANEL BE LOOPEL
 DOME LIGHT JUNCTION BOX TO DOME LIGHT JUNCTION
 BOX FOR EACH SIDE OF A HALLWAY.

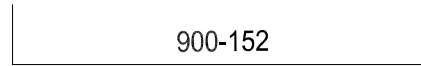

POINTS (1-80)
 900-203
 CONTROL PANEL

SHIELD
 AUDIO
 AUDIO
 ALL CALL
 EMERGENCY LOW
 GROUND
 ROOM COMMON
 + 24 FLD

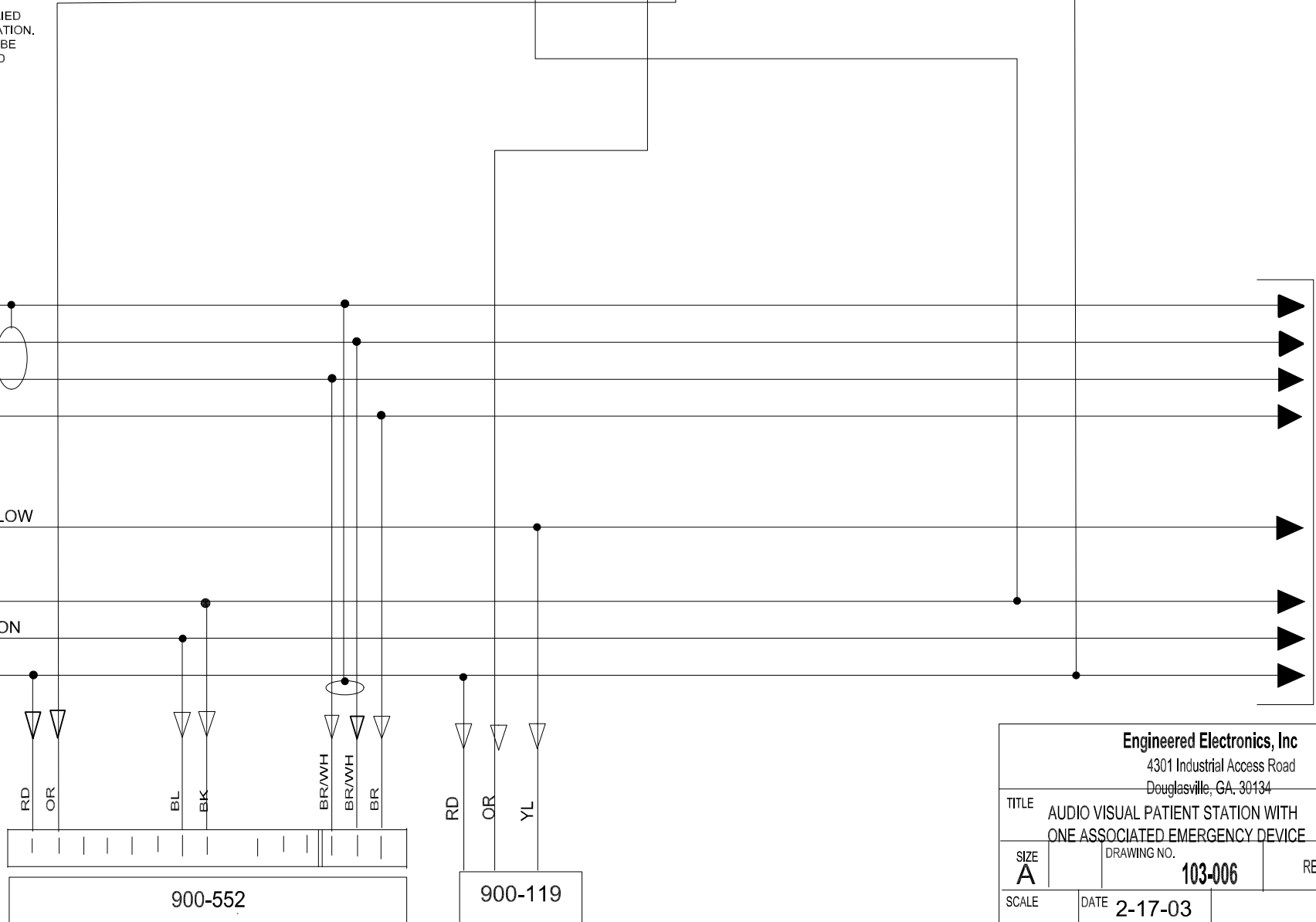

FIELD CONNECTIONS
 900-203 CONTROL PANEL

Engineered Electronics, Inc 4301 Industrial Access Road Douglasville, GA. 30134		
TITLE AUDIO VISUAL PATIENT STATION WITH ONE ASSOCIATED EMERGENCY DEVICE		
SIZE A	DRAWING NO. 103-006	REV 10-04-05
SCALE	DATE 2-17-03	