

Operation of 900-545, 900-552, 900-555 Audio/Visual Patient Station

1. Depressing call switch on call cord or removing plug from station jack will:
 - (a) illuminate associated Red Call LED.
 - (b) illuminate Dome Light.
 - (c) illuminate room indicator button on Master.
 - (d) cause slow intermittent tone to sound at Master and Duty Stations.

2. Answering a call from the Nurse Station Master, when remote call cancel jumpers are in the 1 and 2 positions.
 - (a) Depressing Red Call Led on Master will turn button green indicating connection.
 - (b) Green monitor LED will illuminate on Patient Station.
 - (c) Red call LED on station will be extinguished.
 - (d) Dome Light will be extinguished.
 - (e) Intercom (2-way audio) will be established.
 - (f) Depressing green room button on Master will clear call.

3. Answering a call from the Nurse Station Master, when remote call cancel jumpers are in the 2 and 3 positions.
 - (a) Depressing red call LED on Master will turn button green indicating connection.
 - (b) Green monitor LED will illuminate on Patient Station.
 - (c) Red call LED on station will remain on.
 - (d) Dome Light will remain on.
 - (e) Intercom audio will be established.
 - (f) Depressing green room button on Master will turn it back to red and all call tones will continue to sound. **Call must be cancelled at place of origin.** To clear call, push reset button on Patient Station.

4. Answering a call at Patient Station.
 - (a) Depressing reset switch on station will cancel patient's call and extinguish all lamps.

5. Activation of an Emergency Device that is associated with Patient Station will:
 - (a) light Emergency Dome.
 - (b) override but not cancel patient call.
 - (c) flash red LED room button and emergency indicate button on Master. (Note: If Master room button on Master is in the green answer mode, only the emergency indicate button will flash – return all green buttons back to red when emergency is activated).
 - (d) **All emergency calls have to be cleared at point of origin.**

6. Activation of All Call at Master will:
 - (a) light green monitor LED
 - (b) activate audio path.