


FIELD CONNECTION WIRES-----18 GAGE
 POINT WIRES-----22 GAGE
 STATION WIRES-----22 GAGE

NOTE!

RUNS OVER 1000 FT INCREASE FIELD CONNECTION WIRES TO 16 GAGE.

A CONNECTOR IS SUPPLIED WITH EACH PATIENT STATION. WIRES NOT USED MUST BE ELECTRICALLY ISOLATED

IT IS RECOMMENDED THAT A FIELD CONNECTION CABLE RUN FROM THE CONTROL PANEL BE LOOPED DOME LIGHT JUNCTION BOX TO DOME LIGHT JUNCTION BOX FOR EACH SIDE OF A HALLWAY.

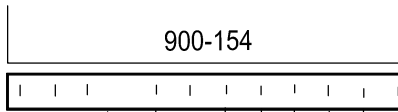

POINTS (1-80)
 900-203
 CONTROL PANEL

EMERGENCY HIGH

EMERGENCY MID


EMERGENCY LOW

GROUND


ROOM COMMON

+ 24 FLD


FIELD CONNECTIONS
 900-203 CONTROL PANEL

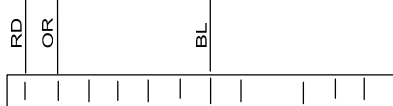


900-535

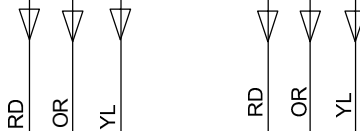

900-119

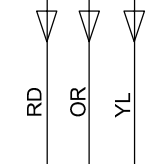

900-112

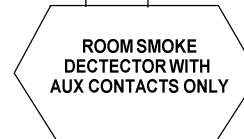

Engineered Electronics, Inc 4301 Industrial Access Road Douglasville, GA. 30134			
TITLE VISUAL PATIENT STATION WITH THREE EMERGENCY DEVICES			
SIZE A	DRAWING NO. 103-002	REV 10-04-05	
SCALE	DATE 02-14-03		