

FIELD CONNECTION WIRES-----18 GAGE
 POINT WIRES-----22 GAGE
 STATION WIRES-----22 GAGE
 AUDIO WIRES-----22 GAGE

NOTE!
 RUNS OVER 750 FT INCREASE FIELD CONNECTION
 WIRES AND POINT WIRES ONE GAGE SIZE.


IMPORTANT

TIE TOGETHER AND TAPE AUDIO SHIELDS AT
 ALL JUNCTIONS POINTS AND CONNECT ONLY
 TO SHIELD (SH) TERMINAL IN CONTROL PANEL.

A CONNECTOR IS SUPPLIED
 WITH EACH PATIENT STATION.
 WIRES NOT USED MUST BE
 ELECTRICALLY ISOLATED

IT IS RECOMMENDED THAT A FIELD CONNECTION
 CABLE RUN FROM THE CONTROL PANEL BE LOOPED
 DOME LIGHT JUNCTION BOX TO DOME LIGHT JUNCTION
 BOX FOR EACH SIDE OF A HALLWAY.

REMOVE TRANSISTOR Q3
 FROM DOMELIGHT PCB
 900-153

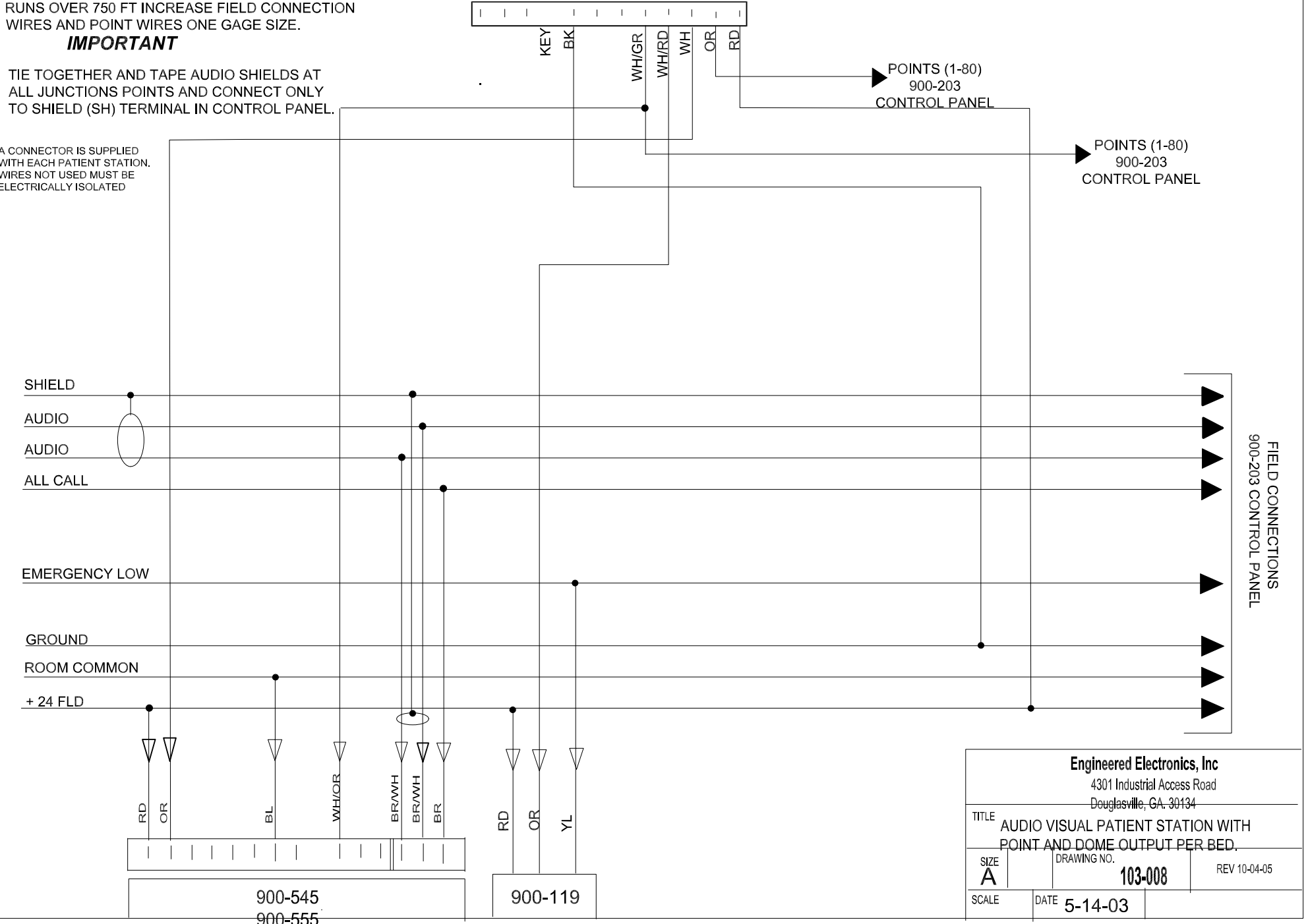

Engineered Electronics, Inc 4301 Industrial Access Road Douglasville, GA. 30134		
TITLE AUDIO VISUAL PATIENT STATION WITH POINT AND DOME OUTPUT PER BED.		
SIZE A	DRAWING NO. 103-008	REV 10-04-05
SCALE	DATE 5-14-03	