

**Engineered Electronics, Inc**

4301 Industrial Access Road, Douglasville, GA. 30134

Two Audio Visual Patient Stations with  
One Associated Emergency Devices

SIZE <b>A</b>	DRAWING NO. <b>103-009</b>	DATE <b>10-3-03</b> REV 6/28/18
------------------	-------------------------------	---------------------------------------

FIELD CONNECTION WIRES-----18 GAGE  
POINT WIRES-----22 GAGE  
STATION WIRES-----22 GAGE  
AUDIO WIRES-----22 GAGE SHEILD

**IMPORTANT**

Tie together and tape Audio Shields at all junction points and connect Only to Shield (SH) terminal in Control Panel.

**NOTE!**

Runs over 750 FT increase field connection wires and Point Wires one gage size.

A Connector is supplied with each Patient Station. Wires not used must be electrically isolated

**900-153**

*It is recommended that a field connection cable run from the control panel be looped dome light junction box for each side of a hallway.*

BK

WH/GR

WH/RD

WH

OR

RD

POINTS (1-80)  
900-203  
CONTROL PANEL

**Remove Q3 from  
Dome Light PCB**

