

Engineered Electronics, Inc

4301 Industrial Access Road, Douglasville, GA. 30134

Three Audio Visual Patient Station with
One Associated Emergency Devices

FIELD CONNECTION WIRES-----18 GAGE
POINT WIRES-----22 GAGE
STATION WIRES-----22 GAGE
AUDIO WIRES-----22 GAGE SHEILD

IMPORTANT

Tie together and tape Audio Shields at all junctions points and connect Only to Shield (SH) Terminal in Control Panel.

NOTE!

Runs over 750 FT increase field connection wires and Point wires one gage size.
A connector is supplied with each Patient Station.
Wires not used must be electrically isolated.

900-154

It is recommended that a field connection cable run from the Control Panel be looped dome light junction box to dome light junction box for each side of a hallway.

SIZE A	DRAWING NO. 103-010	DATE 10/03/03
		REV. 6/29/18

